

V-SHINE[®] FACE MASKS

SINGLE PACKED NON STERILE-DISPOSABLE

EN14683:2005 IIR - ASTM F2101-ASTM F2299

Product description:

non sterile disposable face mask

Dimension: 17.5×9.5cm-6.88"×3.74"

Packaging: 25 pieces per bag

Production date: 2020-04

Product Validity: 24 months

Production Record Certificate:

YCSYJYVJB No.: 202042

Medical Device Registration

Certificate: CXYJB NO.: 202051

FDA Registration Number:


3015532775

Product Executive standard:

YY/T 0969-2013

EN14683:2005 IIR

ASTM F2101-ASTM F2299

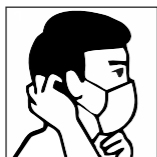

Put the side with the nose clip up (blue side outward) and hang the rubber bands on the ears;



Gently press the nose clip to make it fit the bridge of the nose, and press the ends to follow the cheeks;



Press the nose clip to adjust the lower end of the mask to the proper position of the mandible.

Please ensure that the masks covers from the face to the mandible after deployment for optimum protection.

DO NOT REUSE - THIS IS A SINGLE USE DISPOSABLE PRODUCT.



Storage: Store in a cool, well ventilated place with relative humidity <80%.
Avoid direct sunlight.



6 955702 090689



Manufacturer: Guangdong For You Essentials Manufacturing Co., Ltd. **Address:**
6 Shanhua Road, Yanghe Town, Gaoming District, Foshan City, Tel.: 400-833-7238. **CE-Rep:** Lotus NL BV, Koningin Julianaplein 10, Den Haag, Netherlands